

FACE SHIELD FPM 21

assembly instructions

PRODUCT INFORMATION

It is a protective shield type FPM21 designed to protect the eyes and face against droplets and mild mechanical risks. The shield is not disinfected before shipment. We recommend disinfecting before use. The shield is not certified.


Recommended disinfection methods:

- Hot Air Dryer - 65 °C (149 °F), 60 minutes
- WHO Handrub disinfection - 75% IPA, 5 minutes
- Isopropanol (IPA) - 96%, 5 minutes

Made of:

- visor - PC 0,8 mm thickness
- headband - PETG
- rubber band - 50% ELASTANE, 50% POLYESTER

1.

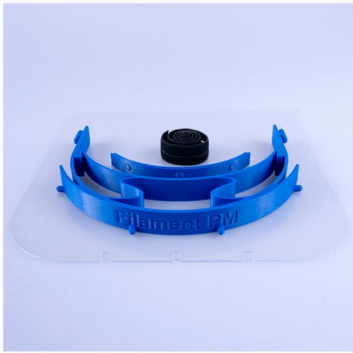

Prepare all the needed.

2.



Remove the protective foils from both sides of the visor.

3.



Put the visor on both middle pins of the headband. Hold and gradually bend both ends of the headband and snap the end holes of the visor onto the end pins.

4.



Turn the shield headband down and snap the lower part of the shield.

5.



Put the rubber band on the ends of the headband. Adjust the length as needed.

6.



Voilà, the shield is done. **Protect yourself and your surroundings.**

Filament 

share your prints with us #filamentpm